

# MELODY Set-Up


## 1. Video Conferencing



**1.1** Turn ON the Monitor by pressing the power button located at the front right bottom.



**1.2** Turn ON the Camera by touching the tablet screen

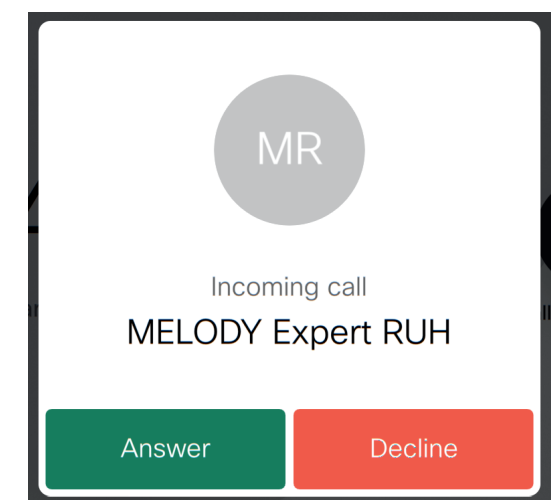

**1.3** Accept the incoming call pressing the Answer button.

## 2. Ultrasound Machine

Refer to diagram on right for item locations

**2.1** Turn ON the Ultrasound Machine by pressing the switch located to the right on the back of the machine.

**2.2** Wait until the screen shown in the diagram appears.

Use the touch screen or mouse to complete the following:

**2.3** Select the 'A' icon located in top left of the screen 

**2.4** Select the Expert site you want to connect to.

**2.5** A small window will appear. Select the option 'Yes'.

**2.6** Select the icon 'DESKTOP SHARING'.


**2.7** Move the cursor to the top center of the screen. A hidden menu will appear.

**2.8** Click the down arrow to open the menu and select the Expert Site you want to connect to.

At this point, the sonographer should have control of the machine.

**2.9** Select the 'A' icon again. 

**2.10** A menu will appear. Select 'Radiologist Supervision' from the three options.

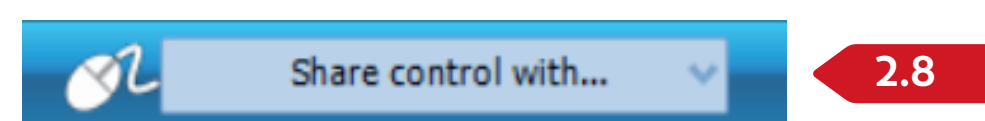

## 3. Robot Control Box

Refer to diagram on right for button locations

**3.1** Turn ON the White Box by pressing the black switch.

**3.2** Press the arrow pointing UP to establish connection.

The two LED lights on the left side should be green

**3.3** Press the round push button.

At this point, the sonographer should have control of the probe.

